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**Put the mask on your face,** with the nose clip on your nose.  
**Position the elastic bands around the back of your head,** on both sides of the ears without crossing them.  
**Check** that the mask covers your face properly (from your nose to your chin).

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**Pinch** the nose clip so that it hugs the shape of your nose.  
**Test the seal.** Refer to the section: *Performance test: check the seal of the mask.*  
**Remove** the mask without touching the front of it, moving it away from your face and clothes.

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The respiratory protection mask FFP2 NR is category III personal protective equipment that complies with the requirements of Regulation 2016/425. It is designed to protect the respiratory health of the user. This product has a filtration efficiency against particles of at least 94%.

**Designation**

- FFP2 NR\* respiratory protection mask. CE 0082\*\* certified. Meets the requirements of European Standard EN 149: 2001+A1: 2009 drafted in relation to Regulation (EU) 2016/425 on personal protective equipment. CE marking in accordance with the requirements of Regulation 2016/425 on personal protective equipment.
- The mask is designated as a "filtering half-mask against particles": PFE\*\*\*>94%, 0.6µm.
- Made in France, at the Combaude site, Clermont-Ferrand (Puy de Dôme).
- You will find the mask documentation (technical sheet, order form, EU declaration of conformity) on our website: protective.michelin.eu**

**Description**

- Respiratory protection mask that folds and is made of non-woven polypropylene.
- Does not contain any components made of latex, PVC or silicone.
- Two elastic loops to go around the head to keep the mask perfectly in place on the user's face.
- A flexible metal nose clip to adjust the mask around the nose for greater comfort and sealing, ensuring a tight fit on the face.

**Precautions**

- USAGE UNIQUE: do not wash the mask.
- Do not use the mask if the packaging is open or damaged
- Contaminated masks must be properly disposed of in accordance with the regulations in your country.
- Discard and replace the mask if it is damaged and beyond the limits set by the working conditions. The mask must not be worn for longer than 8 hours

**Conditioning**

- 1 mask per blister.
- Box of 50 masks, 50 blisters of 1 mask.

**Storage and retention conditions**

Before use, keep the masks in their original packaging, away from contaminated areas, dust, sunlight, extreme temperatures (storage temperature between 5°C and 35°C), excessive humidity and harmful chemicals. The mask has a 2-year shelf life in its sealed plastic pouch from the date of manufacture.

**Instructions prior to use**

- Wash your hands before putting on the mask and after removing it.
- Use this product immediately after opening the packaging.
- Check that the mask is in perfect condition before use (check that the nose clip, elastic loops and the filtration material are in good condition and are not damaged).
- Do not use a mask that is damaged in any way and/or whose storage period has expired.

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\*NR: not reusable / \*\*APAVE SUDEUROPE SAS: notified body code: 0082, responsible for modules B and C2, CS60193 – 13322 MARSEILLE CEDEX 16 – France / \*\*\*PFE: particle filtration efficiency.

MFP Michelin  
 Place des Carmes-Déchaux F-63000 CLERMONT-FERRAND  
 GPS: 45.803701 - 3.124481

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